SCORPION

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*Instruction for use*  
*Istruzioni per l'uso*  
*Instrucciones de empleo*  
*Bedienungsanleitung*

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*Dental scaler tip*  
*Inserti per detartraggio*  
*Insert de detartraje*  
*Scaler spitze*

**Scorpion**

4, rue Enrico Fermi  
63540 Romagnat - FRANCE  
[www.scorpion.fr](http://www.scorpion.fr)

ISO 9001 - ISO 13485

**CE** 0 4 5 9

Fabriqué en France

## Insert de détartrage supragingival - *Supragingival scaling tip*

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**Indications:** The Scorpion scaler tip is perfectly designed for teeth scaling. Due to an improved vibration, power on the scaler equipment can be reduced by the dentist from 20 up to 40 % in comparison with a classical tip. This offers a better comfort for the patient with an unchanged efficiency for the dentist. Screw the tip on the handpiece using the torque wrench. Leave the insert remove the tartar without applying pressure. Maintain irrigation during treatment. The visible wear indicator, a strong but thin titanium coat, can be seen at the sharp end of the tip with its golden colour. The scaler tip is furnished non sterile. Sterilize it before use (autoclave at 134°C for 18 minutes). Do not use on implants and restorative materials. Dispose of the tip according to the applicable regulations regarding medical waste with an infectious risk. **Note:** The use and the application of the product are fully achieved under the responsibility of the dental practitioner. We decline all responsibility in case of bad use of the product.

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