
	<p>Insert de détartrage supragingival - Supragingival scaling tip Supragingival Scaler Spitze - Insert de detartraje supragingival Inserti per detartraggio sopragingivale Υπερουλτρικό ξέστρο αποτρίψωσης</p>												
	<table border="1"> <tr> <td data-bbox="375 140 454 173">BSA</td> <td data-bbox="454 140 777 173">Satelec - NSK Varios - DTE</td> </tr> <tr> <td data-bbox="375 173 454 207">BEM</td> <td data-bbox="454 173 777 207">EMS - Woodpecker</td> </tr> <tr> <td data-bbox="375 207 454 240">BSI</td> <td data-bbox="454 207 777 240">Sirona Sirosonic</td> </tr> <tr> <td data-bbox="375 240 454 274">BKP</td> <td data-bbox="454 240 777 274">Kavo Piezolutax-Sonosoft - MECTRON</td> </tr> <tr> <td data-bbox="375 274 454 308">AIR</td> <td data-bbox="454 274 777 308">Kavo Sonicflex 2000-2003</td> </tr> <tr> <td data-bbox="375 308 454 399">AIRw</td> <td data-bbox="454 308 777 399">W&H Synea-Proxeo NSK Ti-MAX S970-AS2000</td> </tr> </table> <p>Adaptabilité Compatibilité Kompatibel Compatibilidad Compatibilità Συμβατότητα</p>	BSA	Satelec - NSK Varios - DTE	BEM	EMS - Woodpecker	BSI	Sirona Sirosonic	BKP	Kavo Piezolutax-Sonosoft - MECTRON	AIR	Kavo Sonicflex 2000-2003	AIRw	W&H Synea-Proxeo NSK Ti-MAX S970-AS2000
BSA	Satelec - NSK Varios - DTE												
BEM	EMS - Woodpecker												
BSI	Sirona Sirosonic												
BKP	Kavo Piezolutax-Sonosoft - MECTRON												
AIR	Kavo Sonicflex 2000-2003												
AIRw	W&H Synea-Proxeo NSK Ti-MAX S970-AS2000												

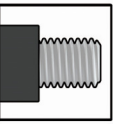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 *Instruction for use*

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Indications: The Scorpion scaler tip is perfectly designed for teeth scaling. The use is restricted to competent and qualified practitioners (dental surgeons, hygienists). Due to an improved vibration, power on the scaler equipment can be reduced by the dentist from 40% in comparison with a classical tip. This offers a better comfort for the patient with an unchanged efficiency for the dentist. Screw the tip on the handpiece using the torque wrench. Leave the insert remove the tartar without applying pressure. Maintain irrigation during treatment. The visible wear indicator, a strong but thin titanium coat, can be seen at the sharp end of the tip with its golden colour. **Disinfection – Cleaning – Sterilization:** The Scorpion tip is supplied non-sterile and must be disinfected, cleaned and sterilized before each use. Only use solutions with proved efficiency for cleaning and disinfection of instruments (e.g. DGHM, FDA or CE marked). 1) Immediately after the invasive procedure, immerse the device in a decontamination bath according to the manufacturer's instructions (concentration, time, temperature). An ultrasonic bath may also be used. 2) Clean the device with running water until free of visible debris and disinfectant residues. 3) Wash the device manually using a soft brush and a cleaning solution according to the manufacturer's instructions (concentration, time, temperature). 4) Rinse thoroughly with running water to remove visible dirt and cleaning product residues. 5) Dry the device with a soft lint-free tissue or with filtered compressed air. 6) Visually check for dirt (and corrosion). 7) Place the device in proper packaging for medical devices sterilization according to ISO 11607 – EN 868. 8) Sterilize the device using autoclave (steam sterilization) at the temperature of 134 °C for 18 minutes. **Note:** Do not use on implants and restorative materials. Dispose of the tip according to the applicable regulations regarding medical waste with an infectious risk. The use and the application of the product are fully achieved under the responsibility of the dental practitioner. We decline all responsibility in case of bad use of the product.

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