 Insert de détartrage supragingival - Supragingival scaling tip Supragingival Scaler Spitze - Insert de detartraje supragingival Inserti per detartraggio sopragingivale - Υπερουλτρικό ξέστρο αποτρίψωσης													
   	<table border="1"> <tr> <td>BSA</td> <td>Satelec - NSK Varios - DTE</td> </tr> <tr> <td>BEM</td> <td>EMS - Woodpecker</td> </tr> <tr> <td>BSI</td> <td>Sirona Sirosonic</td> </tr> <tr> <td>BKP</td> <td>Kavo Piezolutax-Sonosoft - Mectron</td> </tr> <tr> <td>AIR</td> <td>Kavo Sonicflex 2000-2003</td> </tr> <tr> <td>AIRw</td> <td>W&H Synea-Proxeo NSK TI-MAX S970-AS2000</td> </tr> </table>	BSA	Satelec - NSK Varios - DTE	BEM	EMS - Woodpecker	BSI	Sirona Sirosonic	BKP	Kavo Piezolutax-Sonosoft - Mectron	AIR	Kavo Sonicflex 2000-2003	AIRw	W&H Synea-Proxeo NSK TI-MAX S970-AS2000
BSA	Satelec - NSK Varios - DTE												
BEM	EMS - Woodpecker												
BSI	Sirona Sirosonic												
BKP	Kavo Piezolutax-Sonosoft - Mectron												
AIR	Kavo Sonicflex 2000-2003												
AIRw	W&H Synea-Proxeo NSK TI-MAX S970-AS2000												
Adaptabilité Compatibility Κοιμισιbilid Compatibilidad Κοιμισιbilidà Συμβασιòτητα													

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Indications: The Scorpion scaler tip is perfectly designed for teeth scaling. The use is restricted to competent and qualified practitioners (dental surgeons, hygienists). Due to an improved vibration, power on the scaler equipment can be reduced by the dentist from 40% in comparison with a classical tip. This offers a better comfort for the patient with an unchanged efficiency for the dentist. Screw the tip on the handpiece using the torque wrench. Leave the insert remove the tartar without applying pressure. Maintain irrigation during all work. The tip should be replaced when the gold colored Titanium Nitride usage indicator has all worn away. **Pre-disinfection – Cleaning – Sterilization:** The Scorpion tip, and its torque wrench, are supplied non-sterile and must be pre-disinfected, cleaned and sterilized before each use. Only use newly prepared quaternary-ammonium based solutions with proved efficiency for cleaning and disinfection of instruments (e.g. DGHM, FDA or CE marking). 1) Immediately after use, immerse the device in a pre-disinfection bath according to the disinfecting solution manufacturer's instructions (concentration, temperature) respecting a minimum contact time of 5 minutes and rinse with running water. If applicable, separate the tip from its torque wrench. 2) Soak the tip and the key separately in an ultrasonic cleaner bath according to the cleaning solution manufacturer's instructions (concentration, temperature) for a minimum of 15 minutes then rinse with running water. 3) Manually clean the tip and the key separately using a soft brush and a cleaning solution according to the manufacturer's instructions (concentration, temperature) for a minimum of 60 seconds. 4) Rinse thoroughly with running water and then proceed with a final rinse with demineralized water. 5) Dry the devices with a soft lint-free tissue and with filtered compressed air. 6) Visually check for dirt and corrosion. Replace the tip in its torque wrench if applicable. 7) Sterilize using an autoclave (steam sterilization - moist heat) at a temperature of 134°C for 3 - 18 minutes in a packaging appropriate for medical device sterilization. **Note:** Do not use on implants and restorative materials. Dispose of the tip according to the applicable regulations regarding medical waste with an infectious risk. The use and the application of the product are fully achieved under the responsibility of the dental practitioner. We decline all responsibility in case of bad use of the product. Any serious incident occurring in connection with the device must be notified to the manufacturer and to the local competent authority.

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