SCORPION

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Instruction for use

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Dental scaler tip

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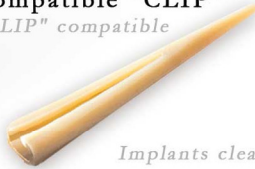
Insert de detartraje

Scaler spitze

Compatible "CLIP"
"CLIP" compatible

Implants cleaning
Maintenance implantaire

Scorpion

4, rue Enrico Fermi

63540 Romagnat - FRANCE

www.scorpion.fr

Fabriqu  en France

Rev/20150130

Insert de détartrage supragingival - *Supragingival scaling tip*

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Indications : The Scorpion-i insert is a dental scaling tip for ultrasonic hand pieces. This patented tip has been designed to maintain optimal vibration even as the tip wears in use. Screw the tip on the handpiece using the torque wrench. When the i-insert is new, adjust your ultrasonic scaler to a 20% power setting, then as the tip wears adjust the power setting according to the Power Setting Table. Please note a higher power setting may cause patient discomfort. The insert will remove the tartar without the need to apply pressure. To avoid too much heat generation good irrigation should be maintained during the treatment. The insert should be replaced when the gold coloured Titanium Nitride usage indicator has all worn away. The Scorpion-i insert is supplied non-sterile. The insert should be sterilized before use (autoclave 134°C, 18 minutes). Do not use on implants and restorative materials. Dispose of the tip according to the applicable regulations regarding medical waste with an infectious risk. **Important:** The use and application of the product are the responsibility of the user. We do not accept any responsibility in the event of misuse.

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