
	Insert de détartrage supragingival - Supragingival scaling tip Supragingival Scaler Spitze - Insert de detartraje supragingival Inserti per detartraggio sopragingivale Υπερουλτρικό ξέστρο αποτρίψωσης
	<b>I-SA</b> Satelec - NSK Varios - DTE
	<b>I-EM</b> EMS - Woodpecker
	<b>I-SI</b> Sirona Sirosonic
	<b>I-KP</b> Kavo Piezolutax-Sonosoft - MECTRON
	<b>I-KA</b> Kavo Piezoled - Piezosoft
	<b>I-AIR</b> Kavo Sonicflex 2000-2003
	<b>I-AIRw</b> W&H Synea-Proxco NSK Ti-MAX S970-AS2000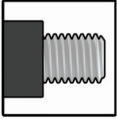

-  **Notice d'utilisation Inserts-i**
-  *Instruction for use*
-  *Bedienungsanleitung*
-  *Instrucciones de empleo*
-  *Istruzioni per l'uso*
-  *Οδηγίες Χρήσης*

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**Indications:** The Scorpion-i insert is a dental scaling tip for ultrasonic handpieces. The use is restricted to competent and qualified practitioners (dental surgeons, hygienists). This patented tip has been designed to maintain optimal vibration even as the tip wears in use. Screw the tip on the handpiece using the torque wrench. When the i-insert is new, adjust your ultrasonic scaler to a 20% power setting or stage 1, then as the tip wears adjust the power setting according to the Power Setting Table. Please note a higher power setting may cause patient discomfort. The insert will remove the tartar without the need to apply pressure. To avoid too much heat generation good irrigation should be maintained during the treatment. The insert should be replaced when the gold coloured Titanium Nitride usage indicator has all worn away. **Disinfection – Cleaning – Sterilization:** The Scorpion tip is supplied non-sterile and must be disinfected, cleaned and sterilized before each use. Only use solutions with proved efficiency for cleaning and disinfection of instruments (e.g. DGHM, FDA or CE marked). 1) Immediately after the invasive procedure, immerse the device in a decontamination bath according to the manufacturer's instructions (concentration, time, temperature). An ultrasonic bath may also be used. 2) Clean the device with running water until free of visible debris and disinfectant residues. 3) Wash the device manually using a soft brush and a cleaning solution according to the manufacturer's instructions (concentration, time, temperature). 4) Rinse thoroughly with running water to remove visible dirt and cleaning product residues. 5) Dry the device with a soft lint-free tissue or with filtered compressed air. 6) Visually check for dirt (and corrosion). 7) Place the device in proper packaging for medical devices sterilization according to ISO 11607 - EN 868. 8) Sterilize the device using autoclave (steam sterilization) at the temperature of 134 °C for 18 minutes. **Note:** Do not use on implants and restorative materials. Dispose of the tip according to the applicable regulations regarding medical waste with an infectious risk. The use and the application of the product are fully achieved under the responsibility of the dental practitioner. We decline all responsibility in case of bad use of the product.

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*Dental scaler tip*  
*Scaler spitze*  
*Insert de detartraje*  
*Inserti per detartraggio*  
*Οδοντιατρικό ξέστρο αποτρίψωσης*

Adaptable - Compatible  
Kompatibel - Compatibilidad  
Compatibilità - Συμβατότητα

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