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Indications: The Scripton® I inser is a dental scaling tip for ultrasonic handpieces. The use is restricted to qualified and qualified practitioners (dental surgeons, hygienists). This patented tip has been designed to maintain optimal vibration even as the tip wears in. Screw the tip on the handpiece using the torque wrench. When the inser is used for the first time, it is recommended to use the torque wrench at 100% of the tip's torque, adjust the torque setting according to the buyer's setting table. Please note a higher torque setting may cause patient discomfort. The inser will remove the tartar without the need to apply pressure. To avoid too much heat generation good irrigation should be maintained during the treatment. The inser should be replaced when the gold coloured Titanium Nitride usage indicates all worn away. **Pre-disinfection — Cleaning — Sterilization:** The scripton tip, and its torque wrench, are supplied non-sterile and must be pre-disinfected, cleaned and sterilized before each use. Only use normal preparations of quaternary ammonium compounds solution with a soft brush and ultrasonic solution according to the manufacturer's instructions (concentration, temperature) for a minimum of 60 seconds. 5) Dry thoroughly with a soft lint-free tissue and then proceed with a final rinsing with demineralized water. 5) Dry the devices with a soft lint-free tissue and with filtered compressed air. 6) Visually check for dirt and corrosion. Replace the tip in its torque wrench if applicable. 7) Sterilize using an autoclave (steam sterilization - moist heat) at a temperature of 134°C for 3 - 18 minutes in a packaging appropriate for medical device sterilization. **Important:** Do not use on implants and restorations. Dispose of the inser in accordance to the applicable regulations regarding medical waste with an infectious route. The use and the application of the product are fully addressed under the responsibility of the dental practitioners. We decline all responsibility in case of bad use of the product. Any serious incident occurring in connection with the device must be notified to the manufacturer and to the local competent authority.

		Insert de détarage supragingival - Supragingival scaling tip Supragingival Scaler Spike - Inset of dental supragingival Inseri per detarage supragingival - Υπεργινιαλ Επορυποιηση; Terminal de descalage supragingival	
		I-SA Satelec - NSK Varios - DTE	
1		I-EM EMS - Woodpecker	
Quantité par conditionnement Quantity per packaging Menge pro Packung Verpackungsgröße Cantidad por embalaje Quantity per pack Flaschenanzahl pro Verpackung Quantità per imballaggio Cantidad por embalajes ボックスあたりの数量		I-SI Sirona Sironic	
Adaptabilité Compatibility Kompatibel Compatibilität Compatibilità Συμβρότητα Compatibilidade 互換性		I-KP Kavo Piezolux-Sonosoft - Mectron	
Verspannungshöhe Spannungshöhe Spannungshöhe Tension height Altitudine de tensión Altezza tensione Altura de tensão Tensión		I-KA Kavo Piezolux - Piezocraft	
Abstand zum Zahn Distance to tooth Abstand zum Zahnschmelz Distanz zum Zahnschmelz Distanza dal dentino Distância ao esmalte Distanza dal dentino		I-AIR Kavo Sonicflex 2000-2003	
Abstand zum Zahn Distance to tooth Abstand zum Zahnschmelz Distanz zum Zahnschmelz Distanza dal dentino Distância ao esmalte Distanza dal dentino		I-AIRw WÄH Synea-Proxeo NSK Tri-MAX 5970-A52000	
Abstand zum Zahn Distance to tooth Abstand zum Zahnschmelz Distanz zum Zahnschmelz Distanza dal dentino Distância ao esmalte Distanza dal dentino		I-LM LM - Amdent	
Tableau de réglage - Power Setting Table			
Regulierungstabelle - Tabla de regla - Quadro di regolazione - Ηλεκτρική ρύθμιση ένστασης			
Quadro di regolazione - 出力（パワー）設定表			
Usure de l'insert - Tip wear		Puissance détarage - Scaler power - Einstellung	
Abnutzung der Spitze		Potencia - Potere - Ένστασης σύγχρονης スケーラーのパワー	
Usure inserto - Desgaste inserto Φθορά Επορυη Desgaste do terminal チップの摩耗		I-SA I-EM I-SI	
Neuf - Unused		I-KP I-KA	
Neu - Nuevo - Νέο - Αρχιμονοτίο Nuevo - 未使用の、新しい		I-AIR I-AIRw	
Usage moyen - Mid-used		I-LM	
mittlere Abnutzung Desgaste medio - Uso media - Μέση χρήση 平均的な使用			
Usure prononcé - Highly used			
starke Abnutzung - Desgaste pronunciado Usura pronunciada - Μεγάλη γηρατοποίηση 使用度の高い			
1		20 %	
2		50 %	
3		100 %	