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**Indications:** The Scripton® I inserter is a dental scaling tip for ultrasonic handpieces. The use is restricted to qualified and qualified practitioners (dental surgeons, hygienists). This patented tip has been designed to maintain optimal vibration levels as the tip wears in. Screw the tip on the handpiece using the torque wrench. When the inserter is new, turn the torque setting to the lower setting (torque 1), then turn the torque setting to the higher setting according to the torque setting in the "Holding Setting Table". Please note a higher torque setting may cause patient discomfort. The insert will remove the tartar without the need to apply pressure. To avoid too much heat generation good irrigation should be maintained during the treatment. The insert should be replaced when the gold coloured Titanium Nitride usage indicates all worn away. **Pre-disinfection — Cleaning — Sterilization:** The scripton tip, and its torque wrench, are supplied non-sterile and must be pre-disinfected, cleaned and sterilized before each use. Only use normal preparations of quaternary ammonium compounds solution with a soft brush and sterilizing solution according to the manufacturer's instructions (concentration, temperature) for a minimum of 60 seconds. 5) Dry the devices with a soft lint-free tissue and with filtered compressed air. 6) Visually check for dirt and corrosion. Replace the tip in its torque wrench if applicable. 7) Sterilize using an autoclave (steam sterilization - moist heat) at a temperature of 134°C for 3 - 18 minutes in a packaging appropriate for medical device sterilization. **Important:** Do not use on implants and restorations. Dispose of the item according to the applicable regulations regarding medical waste with an infectious risk. The use and the application of the product are fully addressed under the responsibility of the dental practitioners. We decline all responsibility in case of bad use of the product. Any serious incident occurring in connection with the device must be notified to the manufacturer and to the local competent authority.

	Insert de détarage supragingival - Supragingival scaling tip Supragingival Scaler Spike - Inset of dental supragingival Inseri per detarage supragingival - Υπεργινια Εστρωματοποιηση; Terminal de descalcificação supragingival 歯肉線上スケーリングチップ					
<b>MD</b>		<b>I-SA</b>	Satelec - NSK Varios - DTE			
	<b>I-EM</b>	EMS - Woodpecker				
	<b>I-SI</b>	Sirona Sironic				
	<b>I-KP</b>	Kavo Piezolux Sonosoft - Mectron				
	<b>I-KA</b>	Kavo Piezolux - Piezocraft				
	<b>I-AIR</b>	Kavo Sonicflex 2000-2003				
	<b>I-AIRw</b>	WAH Synea-Proxeo NSK Tri-MAX 5970-AS2000				
	<b>I-LM</b>	LM - Amdent				
<b>Tableau de réglage - Power Setting Table</b>						
Regulierungstabelle - Tabla de reglae - Quadro di regolazione - Ηλεκτρικού ρύθμισης ένστρωσης						
Quadro di regolazione - 出力（ワバー） 設定表						
<b>Usure de l'insert - Tip wear</b>	Puissance détarageur - Scaler power - Einstellung Potencia - Potencia - Δύναμη ανακυρευτή スケーラーのパワー					
Abnutzung der Spitze	<b>I-SA</b>	<b>I-EM</b>	<b>I-SI</b>	<b>I-AIR</b>	<b>I-AIRw</b>	<b>I-LM</b>
Usure inserto - Desgaste inserto						
Φθορά Εστρωματοποιηση	<b>I-KP</b>	<b>I-KA</b>				
Desgaste do terminal						
チップの摩耗						
<b>Neuf - Unused</b>	20 %					
Neu - Nuevo - Νέο - Αρχηγούμενο Nuevo - 未使用の、新しい	1					
<b>Usure moyenne - Mid-used</b>	50 %					
mittlere Abnutzung	2					
Desgaste medio - Usura media - Μέτρια χρήση						
平均的な使用						
<b>Usure prononcée - Highly used</b>	100 %					
starke Abnutzung - Desgaste pronunciado	3					
Usura pronunciada - Έντονη χρηση						
使用度の高い						