
Adaptable exclusivement sur les inserts Scorpion i - Exclusively compatible with the Scorpion i tips
 Ausschliesslich kompatibel mit allen Scorpion i-Aufsätzen

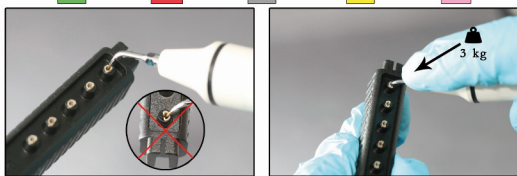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Indications: The CLiP is indicated for the elimination of the plaque and tartar on dental implants and prosthetic restorations. It is designed to work exclusively with the Scorpion i ultrasonic tip and permits cleaning without damaging the fragile surface during scaling.

1) Fixing the CLiP onto the insert
 - Place the tip of the scaler into one of the CLiP's in the CLiPPER (fig. 1)
 - Press the tip firmly into the CLiP (minimum 3 kg) to ensure good fixation (fig. 2)
 - Remove the insert with the CLiP attached and manually check that the CLiP is securely fixed

2) Cleaning implants and restorations (fig. 3)
 - Ensure the power setting on the ultrasonic generator is set from "minimum" to 20% max.
 - Set the water spray to a medium setting
 - When scaling, work with aspiration to ensure that your patient does not swallow any debris
 - When scaling ensure that the CLiP is parallel to the surface being cleaned. Do not press the CLiP against the surface.

3) To remove the CLiP
 - Place the tip with the CLiP into the groove at the end of the CLiPPER (fig. 4)
 - Visualize if the CLiP is worn thanks to the witness line (fig. 5 and 6)
 - The CLiP can be removed from the tip by pulling hard against the wear guide at the side of the CLiPPER (fig. 4)

4) Sterilisation
 - The CLiP and the CLiPPER (as well as the insert and the dynamometric key) are supplied non-sterile and must be disinfected, cleaned and sterilised before each use
 - They must be sterilised at 134° for 18 minutes

5) Elimination
 - The life of the CLiP depends upon the time it is in use.
 - Dispose of the CLiP according to the applicable regulations regarding medical waste with an infectious risk

Important : The CLiP is designed to work exclusively with Scorpion i-inserts (I-SA, I-EM, I-SI, I-KP and I-KA). Using the CLiP with any other tips will result in the CLiP falling off when in use.
 When fixing the CLiP to the tip only use the CLiPPER to ensure secure fixation. Do not try to fix a CLiP to the tip without the CLiPPER for example by hand. The correct use and application of the CLiP is the responsibility of the user. Scorpion does not accept any responsibility in any case of misuse.

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