
Adaptable exclusivement sur les inserts Scorpion-i
 Exclusively compatible with the Scorpion-i tips
 Ausschliesslich kompatibel mit allen Scorpion-i-Aufsätzen
 Adaptable exclusivamente en los inserts Scorpion-i
 Applicabile esclusivamente sugli inserti Scorpion-i
 Συμβατό αποκλειστικά με τα ζεύγη Scorpion-i

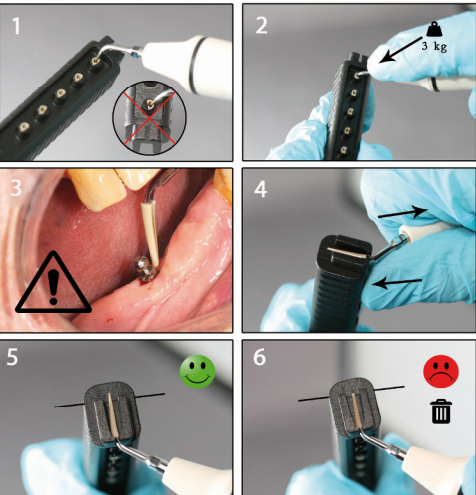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Indications: The CLiP is indicated for the elimination of the plaque and tartar on dental implants and prosthetic restorations. The use is restricted to competent and qualified practitioners (dental surgeons, hygienists). It is designed to work exclusively with the Scorpion-i ultrasonic tip and permits cleaning without damaging the fragile surface during scaling.

Fixing the CLiP onto the insert: Place the tip of the scaler into one of the CLiP's in the CLiPPER (fig. 1). Press the tip firmly into the CLiP (minimum 3 kg) to ensure good fixation (fig. 2). Remove the insert with the CLiP attached and manually check that the CLiP is securely fixed.

Cleaning implants and restorations (fig. 3): Ensure the power setting on the ultrasonic generator is set from "minimum" to 20% max., or Stage 1 for pneumatic handpieces. Set the water spray to a medium setting. When scaling, work with aspiration to ensure that your patient does not swallow any debris and ensure that the CLiP is parallel to the surface being cleaned. Do not press the CLiP against the surface.

To remove the CLiP: Place the tip with the CLiP into the groove at the end of the CLiPPER (fig. 4). Visualize if the CLiP is worn thanks to the witness line (fig.5 and 6). The CLiP can be removed from the tip by pulling hard against the wear guide at the side of the CLiPPER (fig. 4).

Disinfection - Cleaning - Sterilization: The CLiP and the CLiPPER (as well as the tip and the dynamometric key) are supplied non-sterile and must be disinfected, cleaned and sterilized before each use. Only use solutions with proved efficiency for cleaning and disinfection of instruments (e.g. DGHM, FDA or CE marked). 1) Immediately after the invasive procedure, immerse in a decontamination bath according to the manufacturer's instructions (concentration, time, temperature). An ultrasonic bath may also be used. 2) Clean with running water until free of visible debris and disinfectant residues. 3) Wash manually using a soft brush and a cleaning solution according to the manufacturer's instructions (concentration, time, temperature). 4) Rinse thoroughly with running water to remove visible dirt and cleaning product residues. 5) Dry with a soft lint-free tissue or with filtered compressed air. 6) Visually check for dirt. 7) Place in proper packaging for medical devices sterilization according to ISO 11607 - EN 868. 8) Sterilize using autoclave (steam sterilization) at the temperature of 134 °C for 18 minutes.

Elimination: The life of the CLiP depends upon the time it is in use. Dispose of the CLiP according to the applicable regulations regarding medical waste with an infectious risk.

Important : The CLiP is designed to work exclusively with Scorpion i-inserts (I-SA, I-EM, I-SI, I-KP, I-KA, I-AIR, I-AIRw). Using the CLiP with any other tips will result in the CLiP falling off when in use. When fixing the CLiP to the tip only use the CLiPPER to ensure secure fixation. Do not try to fix a CLiP to the tip without the CLiPPER for example by hand. The correct use and application of the CLiP is the responsibility of the user. Scorpion does not accept any responsibility in any case of misuse.

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- Instruction for use
- Bedienungsanleitung
- Instrucciones de empleo
- Istruzioni per l'uso
- Οδηγίες Χρήσης

Nettoyage des implants et restaurations dentaires
 Dental implants and restorations cleaning
 Für die Nachzorge und Reinigung von Zahnimplantaten und deren Aufbauten
 Limpieza de los implantes y restauraciones
 Pulizia di impianti e restauri dentali
 Καθαρισμός εμφυτευμάτων και προσθετικών εργαλείων

Scorpion
 4, rue Enrico Fermi
 63540 Romagnat - FRANCE
 www.scorpion.fr

ISO 13485

CE 0459 (2015)

Fabriqué en France

Rev./ 2016/010