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Indication: Le CLIP est indiqué pour la réalisation de la plaque et tartar sur implants dentaires et prothétiques restaurations. The use is restricted to competent and qualified practitioners (dental surgeons, hygienists). It is designed to work exclusively with the Scopien® ultrasound tip and practice cleaning without damage to the fragile surfaces of the instrument.

Fixing the CLIP onto the insert: Place the tip of the scalpel into one of the CLIP's in the CLIPPER (fig. 1). Press the tip into the CLIP until the tip of the scalpel is held in good fixation (fig. 2). Remove the insert with the CLIP from the CLIPPER and manually clean the fixation ring of the CLIP (fig. 3).

Cleaning implants and restorations: (fig. 3): Ensure the power setting on the ultrasonic generator is set from minimum to 20% (power/duration), (fig. 3) for power ultrasound setting. Set the water spray to a medium stream. When using the CLIP to clean implants and restorations, move the CLIP parallel to the surface being cleaned against the weak side of the side of the CLIP (fig. 4). Visualise the water spray on the CLIP (fig. 5). Remove the CLIP from the CLIPPER and rinse the CLIP under running water against the weak side of the side of the CLIP (fig. 6).

Disinfection - Cleaning - Sterilization: The CLIP, as well as the tips, are supplied non-sterile and must be sterilized by steam sterilization using solutions with proved efficiency for cleaning and disinfection of instruments (e.g. DGHM, FDA or CE marked). Immediately after the invasive procedure, immerse in a disinfectant bath according to the manufacturer's instructions (concentration, time, temperature). After disinfection, remove the CLIP from the CLIPPER and clean the CLIP with a soft brush and restorative materials. Wash manually using a soft brush and a cleaning solution according to the manufacturer's instructions (concentration, time, temperature). 4) Rinse thoroughly with running water to remove visible dirt and debris. Dry the CLIP with a dry cloth. 5) Sterilize the CLIP using an autoclave at 121°C for 15 minutes.

Elbow: (fig. 7) Place the CLIP in standard packaging for medical devices according to ISO 11607 for sterilization. Use autoclave sterilization at the temperature of 134°C for 18 minutes.

Disposal: The CLIP must be disposed of in accordance with the applicable regulations regarding medical waste with an infector risk.

Impression: The CLIP is designed to work exclusively with Scopien Implants I, SIA, I-EM, I-SI, I-KD, I-KA, I-A, I-AR, I-AR-SI. The CLIP must be washed in the CLIP before using it in use.

When fixing the CLIP to the tip only use the CLIPPER to ensure secure fixation. Do not try to fix a CLIP to the tip without the CLIPPER for example by hand. The correct use and application of the CLIP is the responsibility of the user. Scopien does not accept responsibility at any case of misuse. Any serious incident occurring in connection with the device must be reported to the manufacturer and to the competent authority.