# SCORPION

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*Instruction for use*  
*Istruzioni per l'uso*  
*Instrucciones de empleo*  
*Bedienungsanleitung*

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**Indications:** The EP Scorpion scaler tip is perfectly designed for teeth scaling in Periodontology. Its special long end will provide the perfect vibration during deep scaling within perio pockets. Set your scaler unit to Periodontology (**20% max.**). Higher ultrasonic settings may fracture the tip. Screw the tip on the handpiece using the torque wrench. Use the EP Scorpion tip without any manual pressure. Replace the EP Scorpion tip when the golden-colored wear indicator is no more visible. Maintain perio pockets irrigation during treatment to avoid an excessive heating. The scaler tip is furnished non sterile. Sterilize it before use (autoclave at 134°C for 18 minutes). Do not use on implants and restorative materials. Dispose of the tip according to the applicable regulations regarding medical waste with an infectious risk. **Note:** The use and the application of the product are fully achieved under the responsibility of the dental practitioner. We decline all responsibility in case of bad use of the product.

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