**Insert de détartrage sous-gingival - Subgingival scaling tip**  
**Subgingivale Ultraschall-Spitze - Insert de détartrage paro**  
**Inserti per detartraggio subgengivale - Υποούλικα έξτρω αποτρύπωσης**  
**龈下工作尖 - Subgingivalini apnašū šalinimo antgalis**

**MD**

**Dispositif médical**  
**Medical device**  
**Medizinprodukt**  
**Producto sanitario**  
**Dispositivo medico**  
**器械 治療器**  
**醫藥裝置**  
**Medizinische Instrumente**

Adaptabilité	EPSA	Satelec - NSK Varios - DTE
Compatibility	EPEM	EMS - Woodpecker
Kompatibel		
Compatibilidad		
Compatibilità		
Συμβρότητη	EPSI	Sirona Sirosonic
兼容性		
Sunderinammas	EPKP	Kavo Piezolux Sonosoft MECTRON
	EPML	LM - Amdent

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**Indications:** The Scorpion scalar tip is perfectly designed for teeth scaling in Periodontology. The use is restricted to competent and qualified practitioners (dental surgeons, hygienists). Its special long end will provide the perfect vibration during treatment with sub-gingival pockets. Set your scalar unit to periodontics (max. 20%). Higher ultrasonic settings may fracture the tip. Screw the tip on the handpiece using the torque wrench. Use the EP Scorpion tip without any manual pressure. Replace the EP Scorpion tip when the golden-colored wear indicator is no more visible. Maintain perio pocket irrigation during treatment and avoid an excessive heating. **Disinfection – Cleaning – Sterilization:** The Scorpion tip is supplied with a cleaning and disinfection solution (e.g. DGHM, FDA or marked CE). 1) Immediately after the invasive procedure, immerse the device in a decontamination bath according to the manufacturer's instructions (concentration, time, temperature). An ultrasound bath may also be used. 2) Clean the device with running water until free of visible debris and disinfectant residues. 3) Wash the device manually using a soft brush and a cleaning solution according to the manufacturer's instructions (concentration, time, temperature). 4) Rinse thoroughly by running water to remove visible dirt and cleaning product residues. 5) Dry the device with a soft lint-free tissue or with filtered compressed air. 6) Visually check for dirt (and corrosion). 7) Place the device in proper packaging for medical devices sterilization according to ISO 11607 - EN 868. 8) Sterilize the device using autoclave (steam sterilization) at the temperature of 134 °C for 3 - 18 minutes. **Note:** Do not use on implants and restorative materials. Use of the tip according to the applicable regulations regarding medical waste with an infectious risk. The use and the application of the product are fully achieved under the responsibility of the dental practitioner. We decline all responsibility in case of bad use of the product.

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**Indications:** The EP Scorpion scalar tip is perfectly designed for teeth scaling in Periodontology. The use is restricted to competent and qualified practitioners (dental surgeons, hygienists). Its special long end will provide the perfect vibration during treatment with sub-gingival pockets. Set your scalar unit to periodontics (max. 20%). Higher ultrasonic settings may fracture the tip. Screw the tip on the handpiece using the torque wrench. Use the EP Scorpion tip without any manual pressure. Replace the EP Scorpion tip when the golden-colored wear indicator is no more visible. Maintain perio pocket irrigation during treatment and avoid an excessive heating. **Disinfection – Cleaning – Sterilization:** The Scorpion tip is supplied with a cleaning and disinfection solution (e.g. DGHM, FDA or marked CE). 1) Immediately after the invasive procedure, immerse the device in a decontamination bath according to the manufacturer's instructions (concentration, time, temperature). An ultrasound bath may also be used. 2) Clean the device with running water until free of visible debris and disinfectant residues. 3) Wash the device manually using a soft brush and a cleaning solution according to the manufacturer's instructions (concentration, time, temperature). 4) Rinse thoroughly by running water to remove visible dirt and cleaning product residues. 5) Dry the device with a soft lint-free tissue or with filtered compressed air. 6) Visually check for dirt (and corrosion). 7) Place the device in proper packaging for medical devices sterilization according to ISO 11607 - EN 868. 8) Sterilize the device using autoclave (steam sterilization) at the temperature of 134 °C for 3 - 18 minutes. **Note:** Do not use on implants and restorative materials. Use of the tip according to the applicable regulations regarding medical waste with an infectious risk. The use and the application of the product are fully achieved under the responsibility of the dental practitioner. We decline all responsibility in case of bad use of the product.

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