 <p>Insert de détartrage sous-gingival - Subgingival scaling tip Subgingivale Ultraschall-Spitze - Insert de detartraje paro Inserti per detartraggio subgingivale - Υποοολικό Ξέστρο αποτάρτρωσης 靦下工作尖 - Subgingivalinių apnašų šalinimo antgalis</p>																			
 <p>Dispositif médical Medical device Medizinprodukt Producto sanitario Dispositivo médico Ιατρικό συσκευή 醫療裝置 Medicinos priemonė</p>	<table border="1"> <tr> <td>Adaptabilité</td> <td>EP/SA</td> <td>Satelec - NSK Varios - DTE</td> </tr> <tr> <td>Compatibilité</td> <td>EP/EM</td> <td>EMS - Woodpecker</td> </tr> <tr> <td>Compatibilidad</td> <td>EP/PSI</td> <td>Sirona Sirosonic</td> </tr> <tr> <td>Συμβατότητα</td> <td>EP/KP</td> <td>Kavo Piezolux-Sonosoft</td> </tr> <tr> <td>兼容性</td> <td>EP/LM</td> <td>MECTRON</td> </tr> <tr> <td>Suderinamumas</td> <td></td> <td>LM - Amdent</td> </tr> </table>	Adaptabilité	EP/SA	Satelec - NSK Varios - DTE	Compatibilité	EP/EM	EMS - Woodpecker	Compatibilidad	EP/PSI	Sirona Sirosonic	Συμβατότητα	EP/KP	Kavo Piezolux-Sonosoft	兼容性	EP/LM	MECTRON	Suderinamumas		LM - Amdent
Adaptabilité	EP/SA	Satelec - NSK Varios - DTE																	
Compatibilité	EP/EM	EMS - Woodpecker																	
Compatibilidad	EP/PSI	Sirona Sirosonic																	
Συμβατότητα	EP/KP	Kavo Piezolux-Sonosoft																	
兼容性	EP/LM	MECTRON																	
Suderinamumas		LM - Amdent																	

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Indications: The EP Scorpion scaler tip is perfectly designed for teeth scaling in Periodontology. The use is restricted to competent and qualified practitioners (dental surgeons, hygienists). Its special long end will provide the perfect vibration during deep scaling within perio pockets. Set your scales unit to Periodontology (**20% max.**). Higher ultrasonic settings may fracture the tip. Siew the tip on the handpiece using the torque wrench. Use the EP Scorpion tip without any manual pressure. Replace the EP Scorpion tip when the golden-colored wear indicator is no more visible. Maintain perio pockets irrigation during treatment to avoid an excessive heating. **Disinfection – Cleaning – Sterilization:** The Scorpion tip is supplied non-sterile and must be disinfected, cleaned and sterilized before each use. Only use solutions with proved efficiency for cleaning and disinfection of instruments (eg. DGHM, FDA or CE marked). 1) Immediately after the invasive procedure, immerse the device in a decontamination bath according to the manufacturer's instructions (concentration, time, temperature). An ultrasonic bath may also be used. 2) Clean the device with running water until free of visible debris and disinfectant residues. 3) Wash the device manually using a soft brush and a cleaning solution according to the manufacturer's instructions (concentration, time, temperature). 4) Rinse thoroughly with running water to remove visible dirt and cleaning product residues. 5) Wash the device with a soft lint-free tissue or with filtered compressed air. 6) Visually check for dirt (and corrosion). 7) Place the device in proper packaging for medical device sterilization according to ISO 11607 - EN 868. 8) Sterilize the device using autoclave (steam sterilization) at the temperature of 134 °C for 3 - 18 minutes. **Note:** Do not use on implants and restorative materials. Dispose of the tip according to the applicable regulations regarding medical waste with an infectious risk. The use and the application of the product are fully achieved under the responsibility of the dental practitioner. We decline all responsibility in case of bad use of the product. Any serious incident occurring in connection with the device must be notified to the manufacturer and to the local competent authority.

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