	<b>Insert de détartrage sous-gingival - Subgingival scaling tip</b> Subgingivale Ultraschall-Spitze - Insert de detartraje paro Inserti per detartraggio subgingivale Υποουλικό ξέστρο αποτρίγωσης
	<b>Adaptabilité</b> <b>Compatibilité</b> Kompatibel Compatibilidad Compatibilità Συμβατότητα
<b>Quantité par conditionnement</b> <b>Quantity per packaging</b> Menge Verpackungseinheit Cantidad por embalaje Quantità per confezione Ποσότητα ανά συσκευασία	<b>EPESA</b> Satelec - NSK Varios - DTE <b>EPEM</b> EMS - Woodpecker <b>EPSI</b> Sirona Sirosonic <b>EPKP</b> Kavo Piezolutux-Sonosoft <b>EPLM</b> Mectron <b>LM</b> - Amdent

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**Indications:** The EP Scorpion scaler tip is perfectly designed for teeth scaling in Periodontology. The use is restricted to competent and qualified practitioners (dental surgeon, hygienists). Its special long end will provide the perfect vibration during deep scaling within perio pockets. Set your scaler unit to Periodontology (**20% max.**). Higher ultrasonic settings may fracture the tip. Screw the tip on the handpiece using the torque wrench. Use the EP Scorpion tip without any manual pressure. Replace the EP Scorpion tip when the golden-colored wear indicator is no more visible. Maintain perio pockets irrigation during treatment to avoid an excessive heating. **Pre-disinfection – Cleaning – Sterilization:** The Scorpion tip, and its torque wrench, are supplied non-sterile and must be pre-disinfected, cleaned and sterilized before each use. Only use newly prepared quaternary-ammonium based solutions with proved efficiency for cleaning and disinfection of instruments (e.g. DGHM, FDA or CE marking). 1) Immediately after use, immerse the device in a pre-disinfection bath according to the disinfectant solution manufacturer's instructions (concentration, temperature) respecting a minimum contact time of 5 minutes and rinse with running water. If applicable, separate the tip from its torque wrench. 2) Soak the tip and the key separately in an ultrasonic cleaner bath according to the cleaning solution manufacturer's instructions (concentration, temperature) for a minimum of 15 minutes then rinse with running water. 3) Manually clean the tip and the key separately using a soft brush and a cleaning solution according to the manufacturer's instructions (concentration, temperature) for a minimum of 60 seconds. 4) Rinse thoroughly with running water and then proceed with a final rinse with demineralized water. 5) Dry the devices with a soft lint-free tissue and with filtered compressed air. 6) Visually check for dirt and corrosion. Replace the tip in its torque wrench if applicable. 7) Sterilize using an autoclave (steam sterilization - moist heat) at a temperature of 134°C for 3 - 18 minutes in a packaging appropriate for medical device sterilization. **Note:** Do not use on implants and restorative materials. Dispose of the tip according to the applicable regulations regarding medical waste with an infectious risk. The use and the application of the product are fully achieved under the responsibility of the dental practitioner. We decline all responsibility in case of bad use of the product. Any serious incident occurring in connection with the device must be notified to the manufacturer and to the local competent authority.

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